

### CPR MICROMASK®- O2

The **CPR MICROMASK®-O2** features a filtered positive, one-way, non-rebreathing valve that offers low resistance for maximum air flow; one-way valve protects the rescuer whilst its' large opening reduces restrictive 'rescue breathing'. The fold-down / pop-up dome makes for a low profile, easily accessible package that includes a pair of disposable gloves in a durable nylon belt pouch. Ideal for use in CPR training together with the **MDI Training Mouthpiece**.



To achieve a proper face seal and provide for maximum effectiveness, a cuff inflation tube enables you to customise and adjust the shape and firmness of the cushion.

May be used with supplemental oxygen for O2 enriched mouth-to-mask rescue breathing, spontaneous breathing victims and / or as an alternative to a standard oxygen inhalation mask; clear reusable dome allows for patient assessment including lip color and observing for secretions.



Fully assembled, the **CPR MICROMASK®** is conveniently packaged in a zippered belt pouch with a pair of disposable nitrile gloves. Made of materials that remain flexible over a wide range of temperatures, the mask conforms to facial contours and provides an outstanding seal, even under extreme conditions.

The **CPR MICROMASK®** clear plastic dome allows for accurate patient monitoring & may be used on adults, children and infants.

#### Features:

- **Adjustable Air Cushion Seal** easily conforms to the facial contours of adults, children and infants while allowing the user to customise to pillow pressure to suit their individual needs.
- **One-Way Valve with Filter** provides protection for both the responder and the patient.
- **Collapsible Design** opens easily from low-profile storage position to "ready to use".
- **Pre-Assembled** components mean no lost or contaminated parts and no field assembly.
- **Large Valve Opening** allows for greater airflow volume without creating restrictive back pressure for the rescuer.
- **Belt Pouch with Gloves** provides carrying convenience. The durable nylon case features a zipper closer and includes 1 pair of nitrile gloves.
- **Adult, Child and Infant CPR** compatibility is possible by a simple positioning of the nose portion of the mask.

## **CPR MICROMASK® ACCESSORIES:**

### **DISPOSABLE REPLACEMENT VALVES**



The **CPR MICROMASK Replacement Valve** with filter provides the ultimate in protection for both the rescuer and the patient.

Placement of the filter above the one-way, non-rebreathing valve insures no fluids from the patient will degrade the function of the filter.

Designed to offer maximum air flow, the MDI replacement valve is the finest available.

### **DISPOSABLE TRAINING MOUTHPIECES**

**CPR MICROMASK Training Mouthpiece** with filter is compatible with ALL CPR Masks (15 mm I.D. or 22 mm O.D.) and is designed for **manikin training only**

