

Cleaning Procedure: Rainbow® Wraps

Applies to the Rainbow® Wraps (RWI, RWS, RWM, RWL, RWXL and RWXXL)



Items made with fabric, such as the Rainbow Wrap can be hand washed or machine washed in a gentle cycle using a small amount of bleach. Alternatively, the wrap can be washed by itself on the top rack of a dishwasher. This avoids the agitation and spinning of the washer, which can lessen the life of the wrap. Whichever method is used to clean, allow the wrap to air dry. No heat. Do not iron.